Crestal Approach

– Sinus KIT

(HCRSNK)


User Manual

Storage Condition :
Dry place at room temperature(1°C~30°C)

Caution, Consult
accompanying
Documents

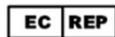OBELIS S.A.
Bd. Général Wahis, 53 1030 Brussels, Belgium
Tel: +32.2.732.59.54

Manufacturer

OSSTEM IMPLANT Co., Ltd.

66-16, Bansong-ro 513beon-gil, haeundae-gu,
Busan, 48002, Republic of Korea

TEL: 82-51-850-2500 FAX: 82-51-861-4693

Crestal Approach - Sinus KIT Instruction for Use - English

■ Product Introduction

HIOSSSEN's Crestal Approach - Sinus KIT (CAS-KIT) is a tool KIT that is optimized for the fast and safe elevation of the maxillary sinus membrane in the Crestal Approach method for the maxillary sinus. Please refer to our product catalog or brochure for detail information.

■ Washing KIT

After performing a dental surgery, wash the tools using the following procedures;

1. During surgery, keep the used tools in a saline solution or distilled water.
2. When the surgery is completed, soak all the tools used in the surgery in alcohol for washing. Caution: Do not wash the tools with hydrogen peroxide. Exposure to hydrogen peroxide may result in the discoloration of the Laser Marking.
3. Wash the tools with distilled or clean flowing water to remove blood stain and other foreign matter.
4. Remove the moisture from the tools and set them in the KIT case according to the coloring coding for easy and correct set-up.
5. After setting the tools, sterilize (132°C for 15minutes) the KIT in an Autoclave and at room temperature.

■ Cautions for Use

1. When using the Ø2.0 Twist Drill, under drill by at least -2mm less than the residual bone appearing in the Panorama or CT image. Use the drill by attaching the appropriate Stopper.
2. Select the CAS-Drill with the optimal diameter in accordance with the diameter and protrusion of the implant from the sinus floor. The recommended speeds are 800 rpm for skilled doctors and 400-600rpm for beginners. Caution: when drilling, do not over-drill by more than 1mm when elevating the membrane
3. When using the CAS-Drill, Depth Gauge, or Bone Spreader coupled with the Stopper, do not apply unnecessarily excessive force to the Stopper when it has reached the desired position.
4. The Depth Gauge is formed with a Tip for the verification of the Membrane elevation. Use the Stopper coupled with the Depth Gauge to check the Membrane elevation. Take care not to do over-drilling by more than 1mm.
5. When using a Hydraulic Membrane Lifter, 1cc Syringe is recommended. Perform the injection using saline water very slowly. Do not use it in case of sinus membrane inflammation or complicated (Septum, etc.) morphology of the sinus floor.
6. The mass of the grafted bone using the Bone Carrier may vary by the Membrane elevation height.
7. Use the Bone Condenser coupled with the Stopper.
8. Do not operate the Bone Spreader at a speed higher than 30rpm. It should be noted that, because it is longer than the tips of the CAS-Drill or Depth Gauge by 2.5mm, when the same stopper is coupled, the tip of the Bone Spreader will protrude into the maxillary sinus by 2.5mm.

9. The Hydraulic Lifter Tube can be reused repeatedly by sterilization in an Autoclave. However, when the sealing function fails because of repeated use, replace it.
10. Keep the tools away from a location where there is a possibility of infection or contamination.
11. The products are medical devices; therefore, they must be sterilized in an Autoclave at 132°C for at least 15 minutes before use.
12. Do not use hydrogen peroxide for sterilization or cleaning, otherwise, the Laser Marking, Anodizing, or Color Coding can be damaged or discolored.
13. The recommended service life cycles for the Ø2.0 Twist Drill, CAS-Drill and Stopper are 50 cycles. However, this may vary depending on the conditions of use.
14. The warranty period of this product is one (1) year from the date of purchase.

■ Disposal

Disposal should be handled in an environmentally sustainable manner according to local regulations. Hazardous waste from contaminated devices or sharps should be disposed of in appropriate containers which meet specific technical requirements.

■ Additional Information for EU Customers

Inform us (HIOSEN) and your competent authority in the case of notice of life-threatening incidents or a severe deterioration in health status related to one of our products.

■ Active-devices capable of combination use

The following applies only to instruments compatible with the active-device (hand piece);

- Shank of instruments for dental hand piece is designed according to type 1 of ISO 1797 to provide full compatibility with all types of standardized instruments.
- Shank of scaler tip for dental scaler hand piece is designed according to ISO 18397 to provide full compatibility with all types of standardized scaler tips.

■ Cleaning and sterilization guideline

Please refer to Cleaning and Sterilization guideline at ifu.osstem.com

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