


# Diagnosis and Removal of Peri-Implantitis

## IM-Cure KIT

- Complete removal of Peri-implantitis
- Minimize damage to the surface of the fixture
- Easy access to Defect site

### Completely Removes Peri-Implantitis with Minimal Damage to the Surface of the Fixture

#### Stage 1 Macro Polishing

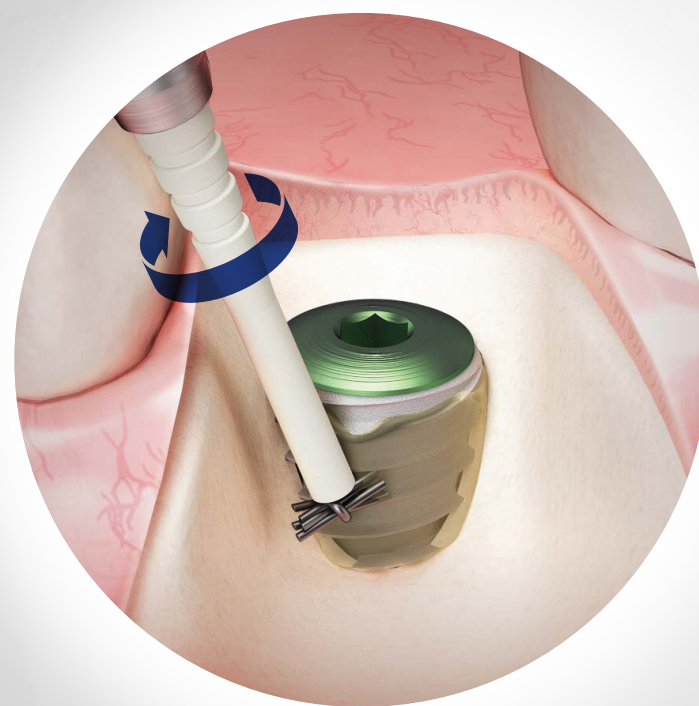


**Smart Brush1**

**Removes more than 95%**

It is designed to fit to the thread of the fixtures in order to achieve good polishing results / the defect site can be easily reached with the brush, thanks to its slim design

#### Stage 2 Fine Polishing

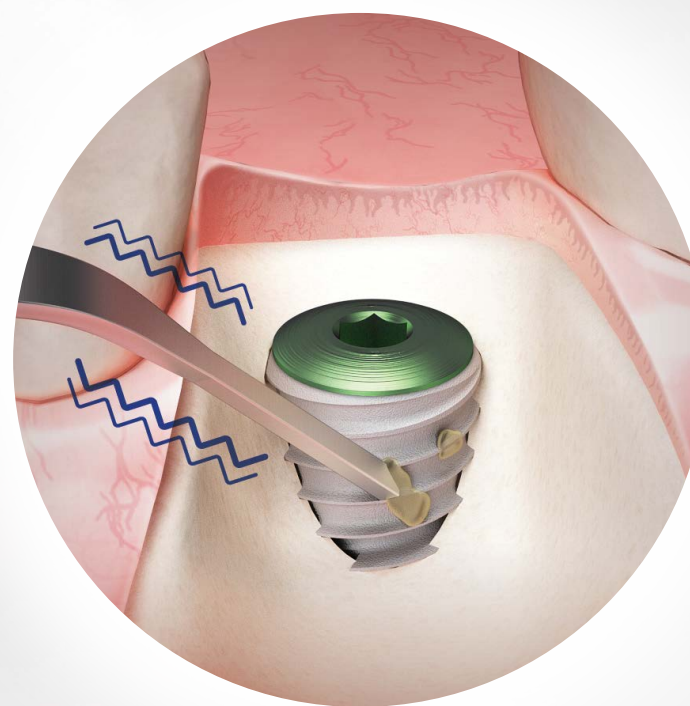

**Scaler Tip**

**Removes the remaining 5%**

Can be freely bent in order to improve accessibility

### Components of IM-Cure KIT

- 1 Probe (2ea)
- 2 Curette (3ea)
- 3 Smart Brush1 (3ea)
- 4 Metal Scaler Tip (3ea)
- 5 Plastic Scaler Tip (3ea)

### Surgical Sequence

#### Removes Peri-Implantitis




1 Probe  
Explore and diagnose peri-implantitis with a Probe



Remove prosthesis and conduct incision on Gingiva



2 Curette  
Remove granulation tissue with Curette

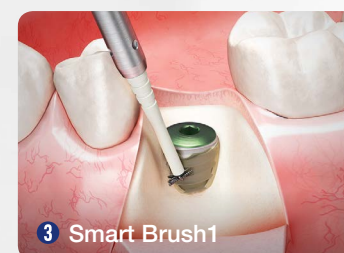


3 Smart Brush1  
Conduct macro polishing with Smart brush1



4 Metal Scaler Tip  
Conduct fine polishing on the implant surface with the metal scaler tip

#### Removal of Peri-Mucositis

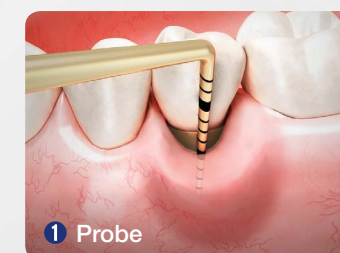

1 Probe  
Diagnose peri-mucositis with a Probe



5 Plastic Scaler Tip  
Conduct polishing of the prosthesis with the Plastic Scaler Tip